Dräger Alcotest[®] 6810

Atemalkohol-Messgerät
Breath Alcohol Monitor

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Contents

For your safety	4
Intended use	5
What's what?	6
Case	6
Measuring instrument	7
Preparation	8
Inserting the batteries	8
Attaching the brief instructions for use	10
Fixing the hand strap	10
Symbols and readings	11
Fitting the mouthpiece	13
Switching on and off	14
Switch on	14
Switch off	14
Automatic switch off	14
Performing the Measurement	15
Conditions	15
Requirements of the person to be tested	15
Automatic measurement	16
Error in delivering the breath sample	18
Removing the mouthpiece	19
Additional possibility for sampling	20
Passive sampling	20
Manual sampling	21

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Dräger Safety AG & Co. KGaA

For your safety

Strictly follow the instructions for use

Any use of the instrument requires full understanding and strict observation of these instructions. The instrument is only to be used for the purposes specified herein. For further information, see the "Technical Handbook Dräger Alcotest® 6810" .

Maintenance

Maintenance must only be performed by experts. We recommend that a service contract be obtained with Dräger Safety and that all repairs be carried out by them. Only authentic spare parts may be used for maintenance. Observe chapter "Maintenance intervals".

Accessories

Only the accessories specified in the order list may be used.

Safe connection with other electrical equipment

Electrical connections to equipment which is not listed in these Instructions for Use should only be made following consultations with the respective manufacturers or an expert.

Liability for proper function or damage

The liability for the proper function of the instrument is irrevocably transferred to the owner or operator to the extent that the instrument is serviced or repaired by personnel not employed or authorized by Dräger Safety or if the instrument is used in a manner not conforming to its intended use. Dräger cannot be held responsible for damage caused by non-compliance with the recommendations given above. The warranty and liability provisions of the terms of sale and delivery of Dräger are likewise not modified by the recommendations given above.

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Intended use

For quick measurement of a person's alcohol concentration through measurement of the breath alcohol concentration. For mobile use with batteries.

Preparation

Dräger Alcotest® 6810 is supplied with standard calibration by the manufacturer.

If the calibration date exceeds the period of 6 months: recalibrate instrument or have it calibrated by Dräger Safety or other authorised organisations.

The date of the last calibration can be found in the quick menu (page 24).

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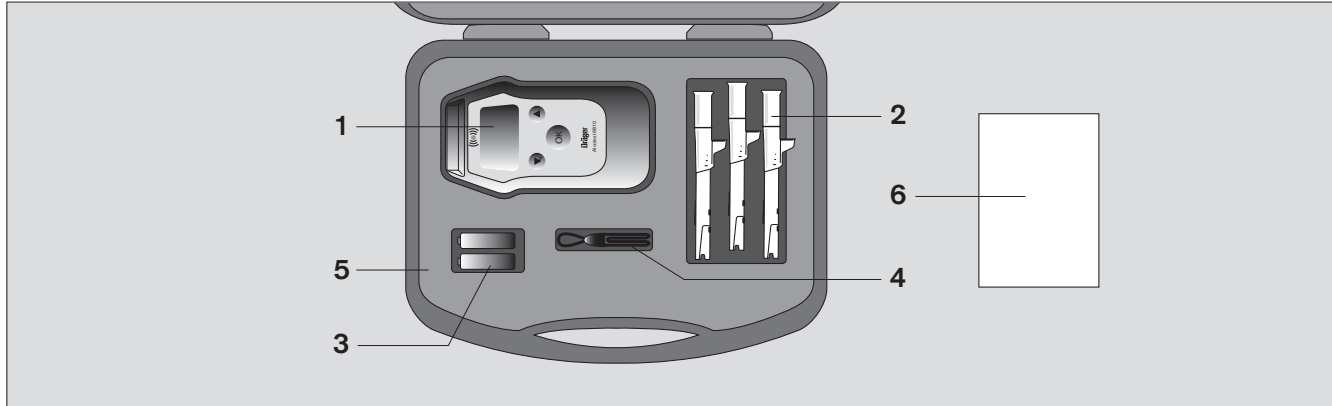
© Alcotest is a registered trademark of Dräger.

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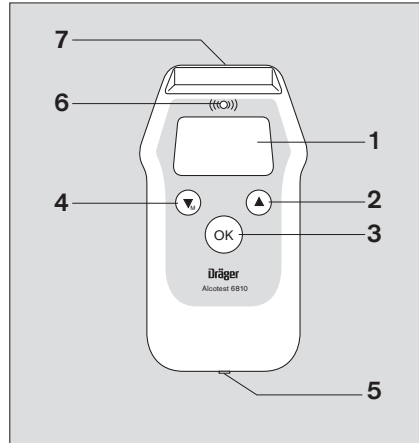
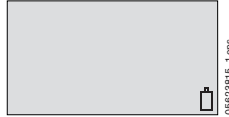
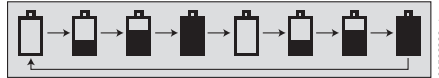
Only for NiMH-batteries

NiMH-batteries can be charged with a special power supply inside of the Alcotest 6810.

- Connect the special power supply (see page 34) with the charging/communication port.
- The charging process is indicated by a pulsing display of the charging symbol.

Caution: Only use the power supply listed in the "Order list" on page 34. Only charge NiMH-batteries.

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
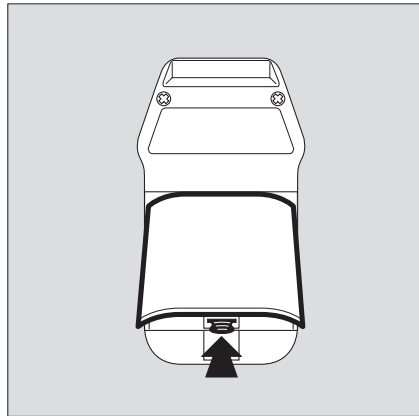
004238115_1.eps

Batteries

- Gently press onto the lock of the battery compartment and remove cover.
- Observe correct polarity when inserting the batteries!
- Close battery cover.

When using NiMH batteries:

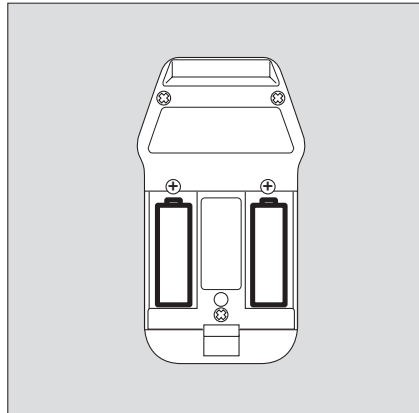- Close special battery cover (see "Order list" on page 34) and fix the screw.

005238115_1.eps

Batteries to be used

2 pieces 1.5 V Type Alkaline (Mignon, LR6, AA)

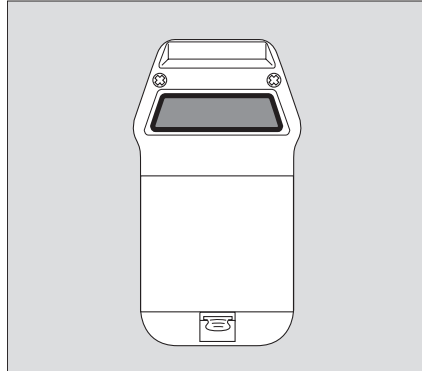
or

rechargeable battery 2 pieces – 1.2 V NiMH

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01023815_1.epa

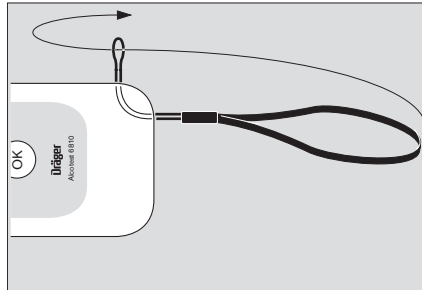
Attaching the brief instructions for use

An area for the "brief instructions for use" sticker is provided on the back of the instrument.

- The instrument is delivered with the brief instructions in four languages.
- Remove sticker from the carrier foil.
- Position the sticker onto the area provided on the back of the housing.

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00923815_1.epa

Fixing the hand strap

- Insert the smaller loop into the hole in the housing.
- Thread the longer loop through the eyelet and pull through.

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Symbols and readings

The measured breath alcohol value is given in mg/L for the basic setting of the Dräger Alcotest® 6810. The instrument can be switched over to all usual measuring units, see the "Technical Handbook Dräger Alcotest® 6810".

Symbols

In these instructions for use several symbols are used corresponding to the messages indicated on the display:

Battery symbol

Indicates the battery charging status:

Full battery

Battery capacity 2/3

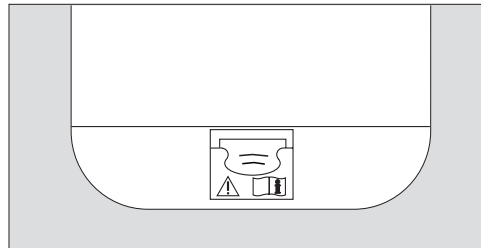
Battery capacity 1/3

Empty battery

Note:

The symbols under the battery flap mean: Attention! Please consult the user instructions! Before changing batteries, please read the instructions.

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Buttons

"OK"-button

Confirms entries or selected functions.
Button to switch instrument on and off.

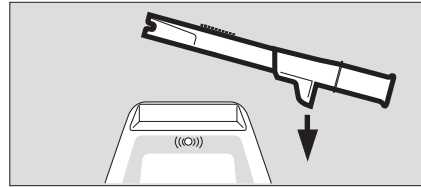
"DOWN" and "UP"

Setting of values and selection of menu functions.

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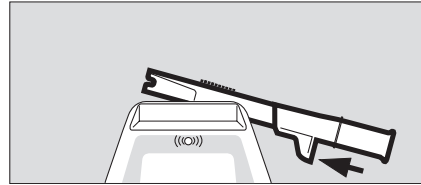
00723815_1.epa

Fitting the mouthpiece

- Open packaging. To keep the mouthpiece hygienic, retain the portion to come in contact with mouth covered by packaging until mouthpiece is correctly fitted to the instrument.
- With the grooved markings facing up, apply the mouthpiece diagonally, shift it upwards and press it into mouthpiece receptacle until audible sound confirms correct engaging.
- Remove packaging completely and commence breath test.

Mouthpiece can be used with orientation to right or left side.

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Switching on and off

Switch on

- Keep  - button pressed for approx. 1 second until start window is displayed. A selftest is carried out. Device settings are displayed.

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If the date for next service is expired:

- Unit has to be calibrated (see "Fault, Cause, Remedy" on page 30).

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Switch off

- Keep  - button pressed.

The instrument switches off after 2 seconds.

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Automatic switch off

The instrument switches off automatically 4 minutes after the last use.

When the batteries are discharged the instrument switches off automatically – the battery symbol is flashing.

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Performing the Measurement

Conditions

- A new mouthpiece is to be used for every person tested.
- Do not apply too high alcohol concentrations to sensor, e.g. by rinsing the mouth with high-grade alcohols shortly before the measurement. By doing so the sensor service life is reduced.
- Keep sufficient distance to the antennas of mobile phones and transmitting stations.

Requirements of the person to be tested

- Maintain a waiting period of at least 15 minutes after drinking alcohol!
Residual alcohol in the mouth can distort the measurement. Aromatic drinks (e.g. fruit juice), mouth sprays containing alcohol, medicines can interfere with measurements. In this case, wait for 15 minutes, too. Rinsing the mouth with water or non-alcoholic drinks does not reduce this waiting period!
- Before sampling the person to be tested must breathe normally. Quick inhaling and exhaling through the mouth must be avoided.
- The person to be tested must be able to provide the required minimum air volume of 1.2 L (factory setting). The breath flow must be constant for a certain minimum blowing period (company setting 2 seconds). Volume and time can be changed via the menu, if necessary.

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Automatic measurement

The sampling is activated automatically after the minimum breath volume and the minimum blowing period are reached.

Preparation

- Insert new mouthpiece in the holder (page 13).
- Switch instrument on (page 14).

After 2 seconds:

- the user info screen (if activated), or
- **"WAIT"** is indicated on the display.
- The user info screen can also be viewed by pressing the  - button. Close with the  - button

After approximately 6 seconds **"READY"** is displayed and a short audible signal is given. At the same time the current test number is indicated. A green light flashes.

The instrument is ready for measurement.

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Atemprobe

Performing the measurement

- Ask the test subject to blow evenly into the mouthpiece without interruption. A sufficient flow is indicated by a continuous tone and a green light.

While the breath sample is provided the display shows: "**Blow**".

When the sampling was successful the green light will cease, the continuous tone will stop and a click will be heard.

The display shows: "**WAIT Analyzing**".

Test result

After 5 to 25 seconds (depending on the instrument temperature and the measured concentration) the measured value is shown on the display.

The value is shown with the respective measuring unit.

The standard setting by the manufacturer is mg/L (milligram ethanol per liter breath air). The setting can be changed via the main menu.

Additionally to the measurement result a signal tone is given, e.g.:

PASS – 1 x, ALERT – 2 x, FAIL – 3 x

Documentation of the result:

While the result is shown on the display, a print-out can be produced with an appropriate Dräger printer.

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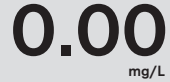
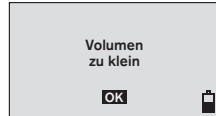
Error in delivering the breath sample

If the sampling volume is too low the display shows: "**Insufficient Volume**"
The horn sounds briefly and a red light flashes.

To repeat press the (OK) - button. The instrument is ready for measurement after 4 seconds.

When exhaling improperly, e.g. by abruptly stopping to exhale or sucking at the end of the sampling, the display shows: "Blow Interruption"
The horn sounds briefly and a red light flashes.

Press the (OK) - button to repeat the measurement. Time period until the instrument is ready for measurement again: 4 to 30 seconds.

Further measurement

- By pressing the (OK) - button a new measurement is prepared.
During the recovery period of the sensor (see "Technical data" on page 31) the display shows: "**WAIT**"

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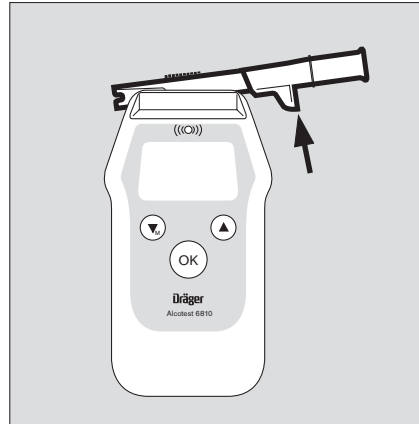
Note:

Switching the instrument off in the meantime does not reduce the waiting period!

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Removing the mouthpiece

- Push the mouthpiece out of the mouthpiece holder.
- Dispose of the mouthpiece in accordance with local regulations.

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ALKOHOL

020238161_L_de.epa

Additional possibility for sampling

The functions can be switched on or off in combination also in the main menu (see page 22).

Passive sampling

During the passive sampling the sampling system is manually activated by the user of the instrument. Possible applications for this function are testing of the ambient air or the exhaled air for alcohol concentrations of persons not able to cooperate (e.g. unconscious persons).

- Hold the **instrument without a mouth-piece** into the air to be analyzed.
- Press the  - button shortly.

Screen 1:

No alcohol available

or:

Screen 2:

In case of alcohol

- Press the  - button for a new measurement.

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
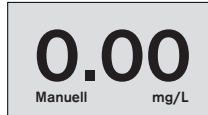
Manual sampling

The sampling can also be activated manually, if the person to be tested is not able to provide the required minimum volume.

Preparation as for the automatic measurement (page 16).

- If "Blow" is indicated on the display after exhaling has started, briefly press the  - button shortly.

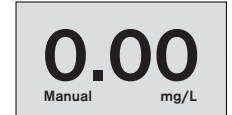When the measured value is displayed "**Manual**" is indicated additionally.

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Menu

Activation

Switch instrument on. When "WAIT" or "READY" appears on the display the menu functions can be called up.

- For quick menu press button  shortly.
- Other menus are not accessible for operators. (see the "Technical Handbook Dräger Alcotest® 6810").

Operation

- Select functions using  and  buttons.
- The selected function is highlighted.
- Activate the selected function by pressing  - button.

- Selection "  **return to measurement**" and pressing the  - button brings the instrument back into the measuring mode.

If no button is pressed within 120 seconds, the instrument automatically returns to the measuring mode.

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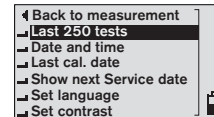
Display of present date and time.

Back with OK - button.

Depending on basic settings of instrument:

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┌ Select language

Defined languages to be chosen in the instrument.

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
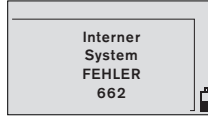
095528B15_1_de.nps

Error

In case of internal error, the following error message is displayed (see page 30).

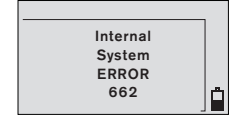If the failure still occurs after the instrument is switched on and off several times, contact Dräger Safety.

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- Check calibration.

Storage

- The Dräger Alcotest[®] 6810 consumes a small amount of current even when switched off. Therefore remove the batteries from the instrument during long periods without use.

Note: In case of long storage periods

Remove batteries from instrument, if the instrument is not used for more than 6 months to avoid damage to the instrument caused by leaking battery acid.

Caution:

Used batteries must not be thrown into open fire. Alkaline batteries must not be recharged or opened by exerting force. Danger of explosion!

- Only dispose of as special waste in accordance with local waste disposal regulations.

For more information about suitable waste disposal companies please contact the local authorities for environmental protection.

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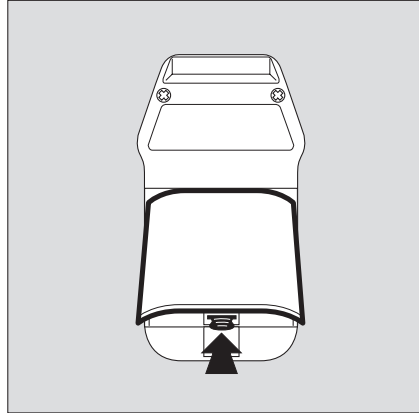
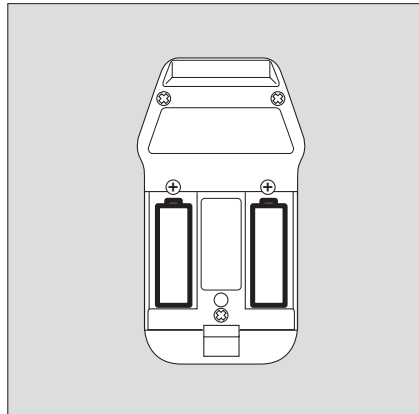
00523815_1.jpg

Cleaning

- Do not submerge in liquid – liquid must not enter the sockets!

To clean:

- wipe with a disposable tissue.
- do not use cleaning agents containing alcohol or solvents.

Changing batteries

If the battery symbol on the display indicates "empty", change the batteries.

Gently press onto the lock of the battery compartment and remove cover.

Remove used batteries and insert new ones – consider correct polarity.

Close battery cover.

Required batteries:

2 pieces 1.5 V Type Alkaline (Mignon, LR6, AA)

or

rechargeable battery 2 pieces 1.2 V NiMH (Mignon, LR6, AA)

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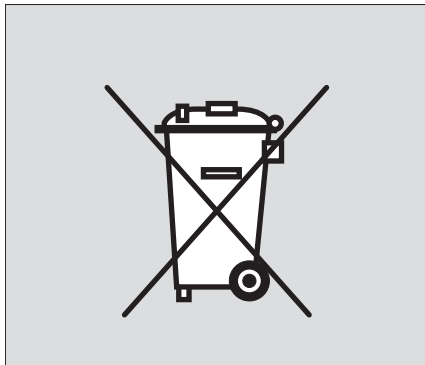
009238165_1.jpg

- For continuity of use it is recommended always to have replacement batteries.

Caution:

Used batteries

- Do not expose to fire,
- Do not recharge,
- Do not force open. Danger of an explosion!

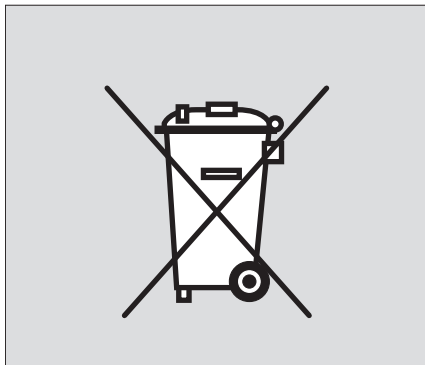
Only dispose of as special waste, in accordance with local waste disposal regulations.

Information can be obtained from environmental agencies and from waste disposal companies.

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009238165_1.jpg

Disposal of the instrument

EU-wide regulations for the disposal of electric and electronic appliances which have been defined in the EU Directive 2002/96/EC and in national laws are effective from August 2005 and apply to this device.

Common household appliances can be disposed of using special collecting and recycling facilities. However, as this device has not been registered for household usage, it must not be disposed of through these means. The device can be returned to your national Dräger Safety Sales Organization for disposal. Please do not hesitate to contact the above if you have any further questions on this issue.

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Fault, Cause, Remedy

Fault	Cause	Remedy
Instrument switches off automatically.	Batteries discharged.	Switch on again. Replace batteries, page 27.
	Instrument has been left ready for measurement for more than 4 minutes.	
Instrument leaves menu-mode automatically and shows measuring mode.	Instrument has an automatically switch off of the menu after 120 sec. without use.	Reselect menu item.
Instrument cannot switched on.	Batteries discharged.	Replace batteries.
Insufficient volume, blowing interruption.	Testperson not blowing hard or evenly enough.	Ask to blow harder and more evenly.
Service date expired.	The service period has expired.	Calibrate by trained service personnel, e. g. from Dräger Safety.
No text in the display.	Instrument error, eventually caused by electrostatic discharge.	Remove batteries (at least 2 sec.) and insert them again, if the fault reappears call Service of Dräger Safety.

Fault in display	Cause	Remedy
"Calibration error"	Calibration not successful.	Calibrate again, by trained service personnel, e. g. from Dräger Safety.
"Accuracy check error"	Accuracy check not successful.	Repeat accuracy check, if fault reappears, call Service of Dräger Safety.
↑ ↑ ↑	Measured concentration exceeds measuring limits.	Use lower concentrations.
"Internal System ERROR" with service code	Instrument error, no measuring function possible.	Remove batteries (2 sec.) and insert them again, if the fault reappears call Service of Dräger Safety.
"Service date expired" unit is blocked	The service period has expired.	Calibrate by trained service personnel, e. g. from Dräger Safety.
"Battery charging error"	No rechargeable NiMH batteries in the unit.	Replace batteries with rechargeable NiMH batteries. Start charging.

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Technical data

Measuring principle	Electrochemical sensor
Measurement range	
Breath alcohol concentration	0.00 bis 2.50 mg/L (Ethanol-Mass per breath volume at 34 °C and 1013 hPa.)
Blood alcohol concentration	0.00 to 5.00 ‰ (Ethanol mass per blood volume resp. mass at 20 °C and 1013 hPa. By giving this data a conversion factor of 2100 between breath and blood alcohol concentrations is used.)
Display range	
	Display range is identical to measuring range. If the result is exceeding the measuring range the display will show three arrows "↑ ↑ ↑" as overflow indication.
Ambient conditions	
For operation	-5 to 50 °C (23 to 122 °F) 10 to 100 % r.h. (non condensing) 600 to 1400 hPa
For storage	-40 to 70 °C (-40 to 158 °F)

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Measurement accuracy

Standard deviation of reproducibility with an ethanol standard

up to 0.50 mg/L ≥ 0.50 mg/L	0.008 mg/L or 1.7 % of measured value whichever value is higher
up to 1.00 ‰ ≥ 1.00 ‰	0.017 ‰ or 1.7 % of measured value whichever value is higher

Sensitivity drift typically 0.6 % of meas. value/month

Intervals before readiness for operation

1st measurement	about 6 s
after a previous measurement with: (breath alcohol concentration)	
0.25 mg/L	20 s
0.50 mg/L	40 s
1.00 mg/L	80 s
> 1.5 mg/L	120 s

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after a previous measurement with:
(blood alcohol concentration)

0.50 ‰	20 s
1.00 ‰	40 s
2.00 ‰	80 s
> 3.00 ‰	120 s

Values for other measuring units can be calculated accordingly
(see page 31).

Dimensions 140 mm x 80 mm x 35 mm (W x H x D)

Weight approx. 195 g

Electrical supply

Batteries 2 pieces 1.5 V Type Alkaline
(Mignon, LR6, AA)
approx. 1500 measurements, or
2 pieces 1.2 V NiMH
(Mignon, LR6, AA)

CE-marking Electromagnetic compatibility
(Directive 89/336/EEC)

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Order list

Name and Description	Part no.
Dräger Alcotest® 6810 (instrument, 3 mouthpieces, 2 batteries, hand strap and case)	83 18 570
Accessories	
Mouth-pieces without non-Return valve (1000 pieces)	68 10 830
Mouth-pieces with non-Return valve (1000 pieces)	68 11 065
Alkaline battery (1 piece)	13 35 804
Rechargeable battery NiMH (1 piece)	83 17 912
Set rechargeable battery , for rechargeable batteries	83 18 565
Technical Handbook Alcotest 6810®	90 23 833
Case for Dräger Alcotest® 6XXX	83 179 20
Leather pouch for Dräger Alcotest® 6XXX (belt fixation)	83 17 911
Leather pouch for Dräger Alcotest® 6XXX (yellow) (Instrument can be operated while staying in the pouch)	83 17 931
"AlcoView" PC-programm for communication with the Alcotest® 6810	83 17 915
PC-connection cable	83 17 593
Power supply for charging NiMH batteries inside of the Alcotest® 6810	83 16 991

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Index

Accessories	34
Alkaline battery supply	33
Ambient conditions	31
Blood alcohol concentration	31
Blowing time	32
Breath alcohol concentration	31
CE-marking	33
Changing Batteries	27
Changing batteries	26
Check calibration	26
Cleaning	27
Dimensions	33
Display range	31
Documentation	17
Electrical supply	33
End of service life	28
Intervals before readiness for operation	32
Maintenance Intervals	26
Measurement accuracy	32
Measurement range	31
Measuring principle	31
Order List	34
Reproducibility	32

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© Dräger Safety AG & Co. KGaA