

AlcoScan 6000i

Sensor Replacement

Overview

The AlcoScan 6000i features the unique ability for the user to replace both the sensor *and* air pathway with a pre-calibrated, ready-to-fit replacement. Replacement requires the removal of just a single screw and is a simple and easily accomplished task that means the Breathalyser is instantly ready to re-use, without having to send it back for re-calibration. The AlcoScan 6000i includes a replacement sensor with the initial package, but additional sensors can be purchased separately at any time from our website www.UKBreathalysers.com

Just follow these four simple steps:


Step One

Remove the sensor screw as shown in the Photograph below and slide the top cover upwards:




Step Two

Slide the existing sensor upwards to remove from the Breathalyser housing:

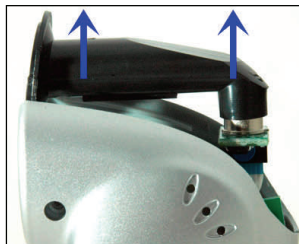


Step Three

Detach the sensor from its connector strip by gently sliding the strip out from the connecting slot:

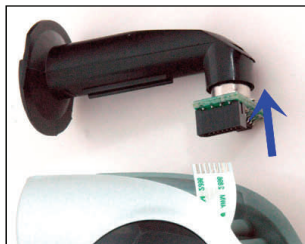

Step Four

Finally, slide the connector strip into the slot in the new sensor with the blue tab facing the blue dot (as shown below) and re-assemble as above.

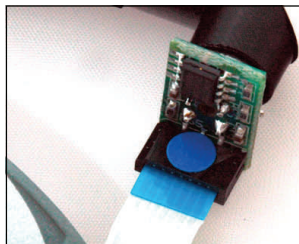

UKBreathalysers.com

Fosters Yard, Forge Lane, Broadbridge Heath West Sussex RH12 3JA

Tel 0870 321 466

Fax 0870 321 4662

Email: enquiries@UKBreathalysers.com